

7555 N. Caldwell Ave. Niles,
IL 60714 800.920.9525



by SEDECAL


STX Pre-Install Sign-Off

Thank you for choosing Vet-Ray by Sedecal for your X-ray needs. This form is important to verify your facility is properly prepared. Please check the boxes indicating you have reviewed each of the corresponding topics and the site is compliant. We recommend you consult with your electrician or contractor when reviewing and completing this form. Please call our Technical Support team if you have any questions.

All orders must have this form completed and returned before Sedecal USA can ship or confirm installation dates.

The X-ray system is normally located on the center of a wall.

Ideal room size is - 9W x 8D x 8H

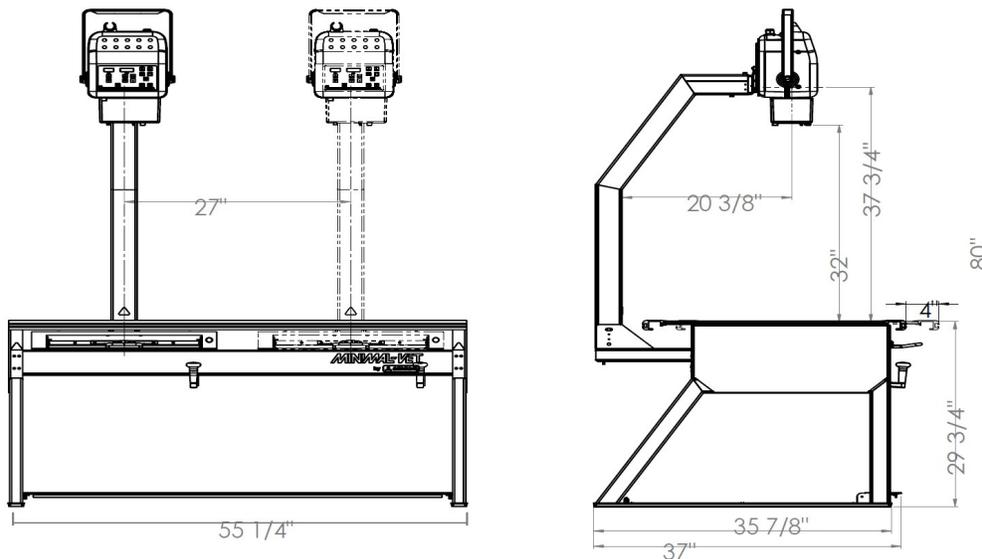

Single Phase - 20 Amp dedicated service

- 4kW requires 110 VAC or 220 VAC on a dedicated 20 AMP service
- 8kW requires 220 VAC on a dedicated 20 AMP service

The dedicated 20 AMP electrical service (Outlet) should be on the back wall and within 5 feet of the center of the table.

For any Technical Support questions, please call: 800.920.9525 ext. 3 and 3 again

Site Preparation

(Please check or initial each box.)

You **MUST** sign and email or fax this completed form to Sedecal USA

Email - orderentry@sedecalusa.com

Fax - 847.394.6966

Electrical

- X-ray room Electrical Service is 20 AMP dedicated as stated on the reverse side.
- A wall mount/stand is provided for the X-ray PC. Please verify space or location

- Completed flooring, ceiling, drywall and paint.
- All cabinets, molding and fixtures are installed.
- HVAC, lighting and electrical outlets are installed and working.
- All door openings into the x-ray room are a minimum of 32" wide. (Ideally 36", if less than 32" please call to discuss).
- Radiation protection, such as lead lined walls or barriers, should be installed as needed according to state or local codes. Also, Lead Aprons available on site.

Please Note

Your IT person must be available for helping with networking and installing the viewing software.

Most digital systems will come with a second workstation which includes a PC and 24" monitor. This is the PACS workstation and it will need some space in the clinic to be placed for use. All images will be transferred to the PACS workstation for viewing. This PACS workstation will need a network connection or a wireless access point for wireless operation on your network. 10 viewing licenses are also part of the package and can be installed on all windows based computers provided by the customer at the time of install.

When the x-ray equipment is delivered, please make sure to inspect the shipping boxes for any damage. Inspect equipment if un-crated by shipping company for any damage. If any damage has been identified to the shipping crate or the equipment, please note on delivery receipt. Also, please make a note on delivery receipt if any damage has been done to your building. If your floors, walls or doorways have been damaged, you **MUST** DOCUMENT and SIGN the delivery receipt stating any damage.

For Digital X-ray Systems

- Acquisition PC (ideally) located in the Radiology area.
- PACS PC and monitor location area are identified (if included)
- Network connections and 110VAC at each PC location.

Do you have a plan for WiFi access on 1 or both PC's?

- YES NO
- Internet access is available and functioning.

Clinic Name

Phone Number

Signature

Title

Print Name

Email Address

Company Purchased From

If the installation engineer arrives on the date scheduled and determines (by this site preparation document) that the site is not ready, the customer will be responsible for the additional cost of re-scheduling the installation engineer once the site is ready. Sedecal USA states these conditions because we do not want the accidental damage to the installed equipment while completing or finishing room construction. Accidental damage of this type and any infiltration of construction dust into the equipment is not covered by the stated equipment warranty.

Sedecal will not take any responsibility for the costs to re-locate the safety disconnect. Sedecal will expect compensation equal to the value of the return trip to complete the installation and will not guarantee date availability.

It is your responsibility to comply with all State and Local shielding requirements and registering the X-ray system with the State of residence. For the most current information on the shielding specifications and contacts for your state, please use the following internet link: <http://www.crcpd.org/mpage/Map>