

## Upgrade to Digital Requirements

Note: Most Digital Upgrades include a 2nd PC for PACS. This 2nd PC and monitor will also require connection to your network. This PACS Server can be accessible to be used as an X-ray viewing station or treated like a server and located in a well ventilated temperature controlled area.

When the digital equipment is delivered, please make sure to inspect the shipping boxes for any damage. If any damage has been identified to the equipment, please make a note on the delivery report.



### Please check or initial each box below

- Please provide make and model of your existing X-ray table or send in a picture.
- A wall mount/stand is provided for the X-ray PC, a 2-3 ft wide wall space will be required for mounting (see image at left).
- The Acquisition PC should be located in the Radiology area.
- Your IT support should be available for help with networking.
- Network connections (Cat5 and 110VAC at each PC location)
- Internet access is available and functioning.

---

**Clinic Name**

**Phone Number**

---

**Signature**

**Title**

---

**Print Name**

---

**Email Address**

#### Please Note

It is your responsibility to have network connections (via Cat5/6) and 110VAC power at each location for the 2 PC's (X-ray acquisition and PACS).

If the installation engineer arrives on the date scheduled and he determines (by this preparation document) that the site is not ready, the customer will be responsible for the additional cost of re-scheduling the installation engineer once the site is ready.

Sedecal will expect compensation equal to the value of the return trip to complete the installation and will not guarantee date availability.

**Please sign and fax or email this completed form to Sedecal USA**

**Fax**  
847.394.6966

**Email**  
orderentry@sedecalusa.com